Infection Prevention and Control Cleaning and Disinfection Instructions for Eye Protection

(developed by the Horizon Health Network)

The following document provides guidance to HCWs for the cleaning and disinfection of eye protection for reuse. There is not a one size fits all solution. Each team should devise a method that maintains these principles and is practical for all involved.

All Settings (Clinical and Non-Clinical Areas) will need to establish a process for cleaning and disinfection of eye protection, being reused, as part of extended use practices. Please contact your local Area Infection Prevention and Control (IPC) Team for education and assistance to establish a process that will work for your department.

Eye Protection	Cleaning & Disinfection Instructions*	Notes
Safety Glasses	<ol style="list-style-type: none"> 1. Perform hand hygiene. Don a clean pair of gloves. 2. Use a hospital approved cleaner/disinfectant wipe to clean and disinfect the surfaces of the eye protection. Wipe the interior of the safety glasses (considered cleanest area) and then the exterior. Ensure any visible soiling is removed. 3. If heavily soiled, use a second cleaner/disinfectant wipe to clean and disinfect the surfaces. Wipe the interior of the safety glasses and then the exterior. 4. Ensure all surfaces remain wet for the cleaner/disinfectant's contact time (products' contact times differ). The manufacturer's product label will indicate the specific time required. 5. Remove gloves and perform hand hygiene. 6. If the eye protection's visibility is compromised by residual disinfectant following the contact time, rinse the eye protection with water. Allow to air dry or use clean absorbent towel. 	If safety glasses appear damaged or compromised- discard- do not reuse.
Goggles	<ol style="list-style-type: none"> 1. Perform hand hygiene. Don a clean pair of gloves. 2. Use a hospital approved cleaner/disinfectant wipe to clean and disinfect the surfaces of the eye protection. Wipe the interior of the goggles (considered cleanest area) and then the exterior. Ensure any visible soiling is removed. 3. If heavily soiled, use a second cleaner/disinfectant wipe to clean and disinfect the surfaces. Wipe the interior of the goggles and then the exterior. 4. Use a new cleaner/disinfectant wipe to clean and disinfect the elastic band. 5. Ensure all surfaces remain wet for the cleaner/disinfectant's contact time (products' contact times differ). The manufacturer's product label will indicate the specific time required. 6. Remove gloves and perform hand hygiene. 7. If the eye protection's visibility is compromised by residual disinfectant following the contact time, rinse the eye protection with water. Allow to air dry or use clean absorbent towel. 	If goggles appear damaged or compromised- discard- do not reuse.

Eye Protection	Cleaning & Disinfection Instructions*	Notes
<p>Face Shield (Single Use)</p> <p><i>Extended Use of Single Use Face Shields with or without foam headbands should be dedicated to an individual HCW for one shift. HCW should label with their name with permanent marker</i></p>	<ol style="list-style-type: none"> 1. Perform hand hygiene. Don a clean pair of gloves. 2. Use a hospital approved cleaner/disinfectant wipe to clean and disinfect the surfaces of the eye protection. Wipe the interior of the face shield (considered cleanest area) and then the exterior. Ensure any visible soiling is removed. 3. If heavily soiled, use a second cleaner/disinfectant wipe to clean and disinfect the surfaces. Wipe the interior of the face shield and then the exterior. 4. Use a new cleaner/disinfectant wipe to clean and disinfect the elastic band. 5. Ensure all surfaces remain wet for the cleaner/disinfectant's contact time (products' contact times differ). The manufacturer's product label will indicate the specific time required. 6. Remove gloves and perform hand hygiene. 7. If the eye protection's visibility is compromised by residual disinfectant following the contact time, rinse the eye protection with water. Allow to air dry or use clean absorbent towel. 8. At end of shift dispose of single use face shield. 	<p>If face shield appears damaged or compromised, do not reuse.</p>
<p>Storage of reusable eye protection:</p>	<ol style="list-style-type: none"> 1. PPE should be stored in a manner to avoid contamination or damage. 2. Clean eye protection must be clearly labelled. 3. The process for storage will be unit/clinical area specific. Follow up with IPC to determine a process 4. There is not a one size fits all solution. Each team should devise a method that maintains these principles and is practical for all involved. 	

*Adapted from Nova Scotia Health Authority: Cleaning and Disinfection Instructions for Face/Eye Protection. April 26, 2020.

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